

ANTIMICROBIAL CATHETER LOCK SOLUTION

C€0123

TauroLock



Indication:

TauroLock[™] is approved to be instilled into catheter-based devices for central venous access in order to prevent CRBSI and biofilm formation. TauroLock[™] maintains patency without undesired side-effects.

Ingredients / Specification:

TauroLock[™] contains anticoagulant and antimicrobial substances. Active ingredients in TauroLock[™] are (cyclo)-taurolidine and citrate (4%). Other components include water for injection. The product is sterile filter processed and supplied as a clear, sterile, non-pyrogenic solution. It is to be used with a port or a catheter-based vascular access device. It is to be instilled in the device lumens between treatments in order to make the internal flow passages resistant to clot formation and hostile to bacterial and fungal growth.

Instillation:

TauroLock[™] is to be instilled into the device lumen between two treatments (e.g. dialysis, oncology, parenteral nutrition).

The solution must be withdrawn prior to initiating the next treatment. If aspiration of TauroLock[™] is not needed or not possible, e.g. in parenteral nutrition, slow flushing of TauroLock[™] prior to the next treatment does not cause systemic effects.

Contraindications:

TauroLock[™] is contraindicated for patients with a known adverse effect to citrate or (cyclo)-taurolidine, or when a patient is currently taking medication with known adverse interaction to citrate or (cyclo)taurolidine.

Storage:

TauroLock[™] must be stored at 15 – 30 °C. Ampoules are for single use only, vials are multi-use and must be used within 48 hours after the first puncture.

article-no.descriptionpackagingTP-01-3TauroLock™ catheter lock solution
3 ml ampoule (single use)10 x 3 mlTP-01-5TauroLock™ catheter lock solution
5 ml ampoule (single use)10 x 5 mlTP-01-10TauroLock™ catheter lock solution
10 ml vial (multi dose)100 x 10 ml

Note: This information does not replace the manufacturer's instruction for use.

Manufacturer:




TauroPharm GmbH August-Bebel-Str. 51 D-97297 Waldbüttelbrunn Tel.: +49 (0) 931 304 299-0

ISO 13485



Im temperature GmbH, German

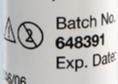

WWW.TAUROLOCK.COM

07.07.2017