



TauroLockTM HEP 100

CE 0123

ANTIMICROBIAL CATHETER LOCK SOLUTION

TauroLockTM HEP 100

CE 0123

Antibiotic
Free

Indication:

TauroLockTM-Hep100 is approved to be instilled into catheter-based devices for central venous access in order to prevent CRBSI and bio-film formation. TauroLockTM-Hep100 maintains patency without undesired side-effects.

Ingredients / Specification:

TauroLockTM-Hep100 contains anticoagulant and antimicrobial substances. Active ingredients in TauroLockTM-Hep100 are (cyclo)-taurolidine, citrate (4%) and heparin (100 IU/mL). Other components include water for injection. The product is sterile filter processed and supplied as a clear, sterile, non-pyrogenic solution.

It is to be used with a port or a catheter-based vascular access device. It is to be instilled in the device lumens between treatments in order to make the internal flow passages resistant to clot formation and hostile to bacterial and fungal growth.

Instillation:

TauroLockTM-Hep100 is to be instilled into the device lumen between two treatments (e.g. oncology, parenteral nutrition).

The solution must be withdrawn prior to initiating the next treatment.

Contraindications:

TauroLockTM-Hep100 is contraindicated for patients with a known adverse effect to (cyclo)-taurolidine, citrate or heparin or when a patient is currently taking medication with known adverse interaction to citrate, heparin or (cyclo)-taurolidine. TauroLockTM-Hep100 is also contraindicated for patients with heparin-induced thrombocytopenia (HIT) or increased bleeding risk.

Storage:

TauroLockTM-Hep100 must be stored at 15 – 30 °C. Ampoules are for single use only.

Note: This information does not replace the manufacturer's instruction for use.



article-no.	description	packaging
TP-03-3	TauroLock TM -Hep100 catheter lock solution 3 ml ampoule (single use)	10 x 3 ml

Manufacturer:

TauroPharm
GmbH

TauroPharm GmbH
August-Bebel-Str. 51
D-97297 Waldbüttelbrunn
Tel.: +49 (0) 931 304 299-0

ISO 13485

Contact Us:

JTS Medcare Solutions (Pty) Ltd

cell: +27 722 178 183

email: info@jtsmedsol.co.za

web: www.jtsmedsol.co.za

WWW.TAUROLOCK.COM

07.07.2017