



# TauroLock<sup>TM</sup> HEP 500

CE 0123

ANTIMICROBIAL CATHETER LOCK SOLUTION

# TauroLock™ HEP 500

CE 0123

Antibiotic  
Free

## Indication:

TauroLock™-Hep500 is approved to be instilled into catheter-based devices for central venous access in order to prevent CRBSI and bio-film formation. TauroLock™-Hep500 maintains patency without undesired side-effects.

## Ingredients / Specification:

TauroLock™-Hep500 contains anticoagulant and antimicrobial substances. Active ingredients in TauroLock™-Hep500 are (cyclo)-taurolidine, citrate (4%) and heparin (500 IU/mL). Other components include water for injection. The product is sterile filter processed and supplied as a clear, sterile, non-pyrogenic solution.

It is to be used with a port or a catheter-based vascular access device. It is to be instilled in the device lumens between treatments in order to make the internal flow passages resistant to clot formation and hostile to bacterial and fungal growth.

## Instillation:

TauroLock™-Hep500 is to be instilled into the device lumen between two treatments (e.g. dialysis).

The solution must be withdrawn prior to initiating the next treatment.

## Contraindications:

TauroLock™-Hep500 is contraindicated for patients with a known adverse effect to citrate or (cyclo)-taurolidine, citrate or heparin or when a patient is currently taking medication with known adverse interaction to citrate, heparin or (cyclo)-taurolidine. TauroLock™-Hep500 is also contraindicated for patients with heparin-induced thrombocytopenia (HIT) or increased bleeding risk.

## Storage:

TauroLock™-Hep500 must be stored at 15 – 30 °C. Ampoules are for single use only, vials must be used within 48 hours after the first puncture.

**Note:** This information does not replace the manufacturer's instruction for use.



article-no.	description	packaging
TP-02-5	TauroLock™-Hep500 catheter lock solution 5 ml ampoule (single use)	10 x 5 ml
TP-02-10	TauroLock™-Hep500 catheter lock solution 10 ml vial (multi dose)	100 x 10 ml

Manufacturer:



TauroPharm GmbH  
August-Bebel-Str. 51  
D-97297 Waldbüttelbrunn  
Tel.: +49 (0) 931 304 299-0

ISO 13485

Contact Us:

**JTS Medcare Solutions (Pty) Ltd**

cell: +27 722 178 183

email: [info@jtsmedsol.co.za](mailto:info@jtsmedsol.co.za)

web: [www.jtsmedsol.co.za](http://www.jtsmedsol.co.za)

[WWW.TAUROLOCK.COM](http://WWW.TAUROLOCK.COM)

07.07.2017